



A Component parts

- 1 CPR quality indication light** The CPR quality indication light illuminates if the correct chest compression depth, speed, release are performed and the correct fingers (or thumbs) position detected.
- 2 Chest compression rate indication light** Indicates the speed /rate of chest compressions performed.
- 3 Chest compression depth indication light** Indicates the depth of chest compressions performed.
- 4 Ventilation indication light** Indicates the ventilation volume performed.
- 5 Power button** Press the power button for approximately 1 second to turn on/turn off the manikin.
- 6 (+) and (-) button** This switches on/off ventilation feedback and facilitates audio and/or lights.
- 7 Bluetooth disconnect button** Terminates the Bluetooth connection. (Brayden Baby Pro only)
- 8 Scenario mode button** Sets the scenario mode. (Brayden Baby Pro only). This switches on the brachial pulse on the left arm.
- 9 Power status indication light** Illuminates green if the battery is normal and flashes red if the battery is low.
- 10 Bluetooth connection status indication light** Illuminates blue if the Bluetooth connection is established.
- 11 Scenario mode indication light** Illuminates blue to indicate that the manikin is operating currently in scenario mode. (Brayden Baby Pro only)
- 12 AC Adaptor connection socket** Allows the optional AC adaptor to be connected.

B Installing and connecting the mobile app

1. Search for the **BRAYDEN ONLINE app** from the **App Store** on the iPad and install it.
2. Turn on the manikin and run the app.
3. Press the **Connect** button on the app and select the desired item to connect.
4. When you press the **Connect** button, the manikin and app will connect and the **Bluetooth connection status indication light** of the manikin will illuminate blue.

(Note) Recommended specifications for iPad
- Operating system : iOS 13 or higher

C What the CPR real time feedback lights indicate

1. Ventilations

- a. The **Ventilation indication light** will illuminate when ventilations are delivered at the correct volume and will flash when an excessive volume is delivered.

2. Chest Compressions

- a. The **Chest compression depth indication light** illuminates according to the depth of compressions delivered. All the lights flash if the depth of the chest compressions are too excessive.
- b. The **Chest compression rate indication light** (which go from the chest to the head) illuminates according to the rate/speed of the compressions delivered. If too fast, then the rate/speed is too fast.
- c. The **CPR quality indication light** only illuminates if the correct chest compression depth, speed, release are performed and the correct fingers (or thumbs) position detected.

(Note)

- When the manikin is turned on, an initial automatic calibration will be carried out for approx. 2 seconds. It is recommended that chest compressions are not performed on the manikin at this time.

(Note) for Brayden Baby Pro only

- When connecting the manikin to the App for assessment, the CPR real time feedback lights will not illuminate.